

# Baxter

## PrisMax

ACUTE CARE SYSTEM

# MAXIMISE NURSING EFFICIENCY WITH THE AUTO-EFFLUENT DRAIN ACCESSORY

powered by the PrisMax system

The **PrisMax** System with the optional Auto-Effluent Drain Accessory eliminates the need to change effluent bags during CRRT, freeing up nursing time and enabling more time for patient care.<sup>1</sup>



## Save Valuable Time

Without the optional Auto-Effluent Drain Accessory, as often as every 1-2 hours, an ICU nurse will have to change effluent bags for each patient on CRRT:

Leaving their current task, to return to the CRRT device

Interrupting therapy during the bag change

Lifting a bag of effluent and emptying it in a nearby drain

Connecting a new bag, and then moving to their next task



## Making Treatment Delivery More Efficient

The unique Auto-Effluent Drain Accessory is designed to reduce treatment interventions, freeing up valuable nursing time.<sup>1</sup>

By eliminating the manual task of changing effluent bags as often as every 1-2 hours, users may **save an average of more than 30 minutes** per patient per day in nursing time.<sup>2,\*</sup>


**Gain up to 30 minutes additional therapy time** for achieving prescription targets by decreasing therapy downtime due to effluent bag changes.<sup>2,\*</sup>

**Bag changes may be reduced by 60%** compared to similar CRRT systems.<sup>2</sup>

\*Based on data from a pilot evaluation of 305 treatments with the **PrisMax** system at eight ICUs in six countries



"The benefit goes beyond the actual number of minutes saved because the nurse can focus on more important things in a complex patient scenario."<sup>3</sup>

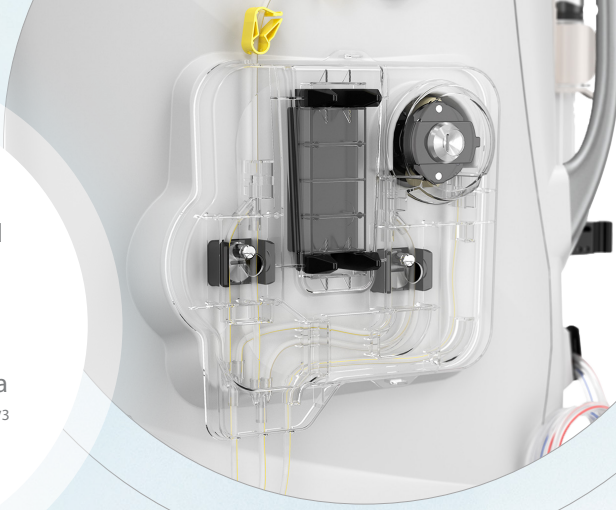

## The Auto-Effluent Drain Accessory


The Auto-Effluent Drain Accessory continuously pumps effluent collected during therapy into a drain, while providing the accuracy of a scale-based measurement system. Eliminating effluent bag changes minimises user exposure to effluent.<sup>1</sup>

Using a hook and line system having up to an 18 m (60 ft) range,\* the Auto-Effluent Drain Accessory works with a range of room layouts and drains.<sup>1,†</sup>

The Auto-Effluent Drain Accessory can remain in place for up to six days, even when changing CRRT therapy modes or blood filter sets.<sup>1</sup>

\*An extension line may be needed depending on the distance between the device and drain

†Disposal should be managed in accordance with your hospital's infection control policy

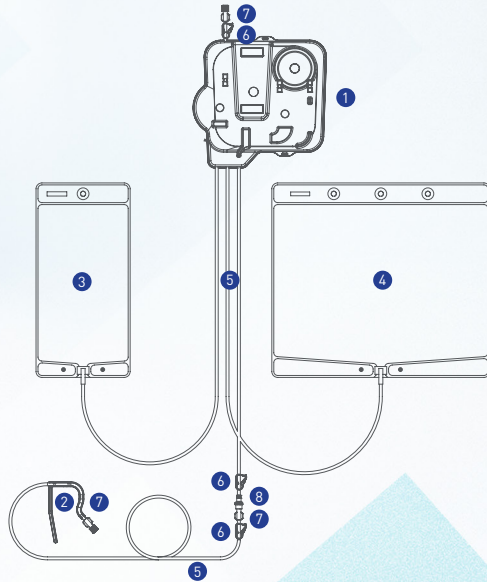

- |                     |                          |
|---------------------|--------------------------|
| 1. Cartridge        | 5. Effluent tube         |
| 2. Drain hook       | 6. Pinch clamps          |
| 3. Effluent bag, 1L | 7. Female luer connector |
| 4. Effluent bag, 5L | 8. Check valve           |

- The Auto-Effluent Drain Accessory eliminates the need to change effluent bags during treatment by alternating effluent fill and drain between the 1 litre and 5 litre bags.
- Effluent flows into one bag while the other bag drains.
- When a bag fills to 1 litre, the pinch valves change position automatically to begin draining that bag and filling the other.
- Effluent draining can be paused as needed to allow for changes in patient location or drain point.
- The one-way check valve at the proximal end of the drain line prevents effluent backflow.

For more information, please speak with a Baxter representative.

For safe and proper use of products mentioned herein, please refer to the appropriate Operators Manual or Instructions for Use.

### References:

1. Baxter – PrisMax Operators Manual. AW8005 Rev A. 2017; Sep. 2. Baxter – PrisMax Limited Controlled Distribution Report #ER6877 Rev A. Data on file. 2018. 3. Broman M, Bell M, Joannes-Boyau O, Ronco C, The Novel PrisMax Continuous Renal Replacement Therapy System in a Multinational, Multicentre Pilot Setting. Blood Purif 2018;;220-227..

Baxter and PrisMax are trademarks of Baxter International Inc. or its subsidiaries.  
ANZ/MG207/18-0004c

AUSTRALIA  
Baxter Healthcare Pty Ltd  
1 Baxter Drive, Old Toongabbie  
NSW 2146, Australia  
1800 Baxter (229 837)  
www.baxterhealthcare.com.au

NEW ZEALAND  
Baxter Healthcare Ltd  
33 Vestey Drive, Mt Wellington,  
Auckland 1060, New Zealand  
0800 Baxter (229 837)  
www.baxter.co.nz